Utopia® NE Full Face Mask

Instructions for Use – 06053 Rev H

Intended Use

The Utopia NE Full Face Mask is intended to provide a patient interface for application of noninvasive ventilation. This mask is to be used as an accessory to ventilators which have adequate alarms and safety systems for ventilator failure and which are intended to administer CPAP or positive pressure ventilation for treatment of respiratory failure, respiratory insufficiency or obstructive sleep apnea. A separate exhalation device and safety valve must be used with this mask.

The Utopia NE is intended for use on adult patients (>30 kg), who are appropriate candidates for noninvasive ventilation, in the hospital/institutional environment only. Dispose of the interface after single patient use.

Specifications


- The deadspace associated with this mask is: SM – 299 cm³, MD – 336 cm³, LG – 392 cm³.
- The maximum unintended leak from parts of the mask is 3.5 LPM at 20 cm H₂O and 5.0 LPM at 35 H₂O.
- Storage Temperature: -20 to 60°C (-4 to 140°F), Operational Temperature: 5 to 40°C (-41 to 104°F), Humidity: 0–95%

Warnings


- In the USA, federal law restricts this mask to sale by, or on the order of, a physician. (Rx ONLY)
- This mask requires a separate exhalation device.
- Do not use this mask in or near magnetic resonance imaging equipment, as it contains metal parts.
- A minimum pressure of 3.0 cm H₂O at the mask (CPAP) is required.
- This mask does not include an anti-asphyxia valve that would allow the patient to breathe if the ventilator failed. The mask should only be worn when the ventilator is turned on and operating properly. This mask must be used with ventilators which have adequate alarms and safety systems for ventilator failure. In the event of ventilator failure, use of this mask requires the same level of attention and assistance as the use of a tracheal tube.
- If the patient experiences skin irritation, consult the physician. The mask consists of silicone, polycarbonate, and polypropylene.
- Contact the physician if the patient experiences unusual chest discomfort, shortness of breath, stomach distention, belching, or severe headache when he/she awakens or when he/she is receiving positive airway pressure.
- To minimize the risk of vomiting during sleep, the patient should avoid eating or drinking for three hours before using the mask. This mask is not recommended if the patient is taking a prescription drug that may cause vomiting.
- This warning applies to most CPAP devices. At low CPAP pressures, the air flow through the vent holes(s) on the exhalation device may not be enough to clear all of the exhaled gas (CO₂) from the mask. The patient may breathe in some of the air that he/she has exhaled. Do not block or otherwise try to seal the vent hole(s) on the exhalation device.
- This mask is not suitable for providing life support ventilation.
- This mask is intended for single-patient use only and thus use by multiple patients is to be avoided due to the risk of transmitting infection. Do not attempt to sterilize or use disinfectants on this mask.

Contraindications


This mask is contraindicated for patients without a spontaneous respiratory drive. This mask may not be suitable for persons with the following conditions: impaired cardiac sphincter function, excessive reflux, impaired cough reflex, and hiatal hernia. It should not be used if patient is uncooperative, obtunded, unresponsive, or unable to remove the mask.

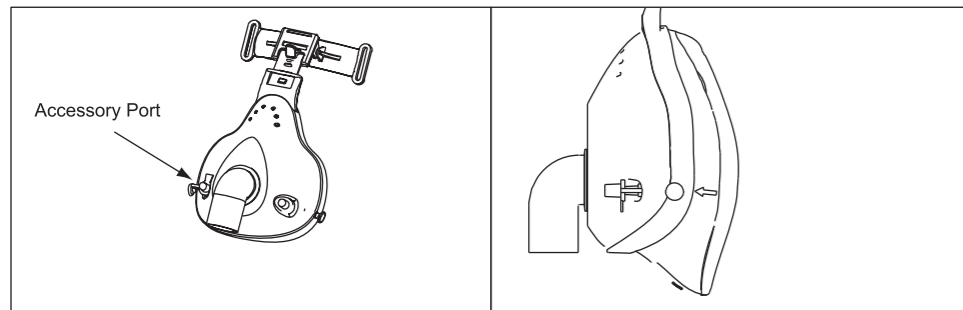**Instructions for Use**

- Prior to use, clean the mask as described in the cleaning instructions.
- Clean and thoroughly dry the patient's face before putting on the mask.
- Attach the headgear to the forehead support by inserting the straps with the Velcro tabs through the slots on the support. The straps should be led with the "hook" side facing outward so they can be folded back and secured against the "loop" surface of the strap.
- Attach the circuit tube from the ventilation system to the elbow. *NOTE: The Utopia NE is designed to accept a 22 mm male connector. If the hose has a female connector, a male-male adaptor can be used between the elbow and the hose.*
- Place the mask over mouth and position the cushion flap under the chin. Pull the headgear back over the top of the head. Adjust the straps on the forehead support until it fits snug against the face.
- Attach the quick-release buckles to the pegs on the mask and pull the buckle back (towards the face) until an audible "click" is heard.

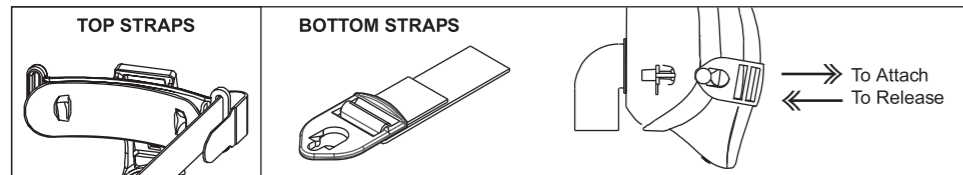**Adjusting and Removing the Mask**

- To adjust the mask, gently pull each headgear strap until the mask sits squarely on the patient's face. NOTE: The seal on the mask cushion is designed to inflate towards the face, so generally only light tension on the straps is required. Over-tightening may cause leaks.
- The height of the forehead brace can be adjusted by pressing on the thumb lever, and moving the brace to a different slot in the support link.
- To adjust cushion pressure on the nose bridge, bend the support link where it exits the mask shell. NOTE: To reduce pressure on the nose bridge, bend the support link forward so that the forehead brace is closer to the face.
- To remove the mask, remove the headgear straps by pressing the tab on the quick release buckles forward, away from the face. Keeping the straps on the forehead support in place, lift the mask over patient's head.

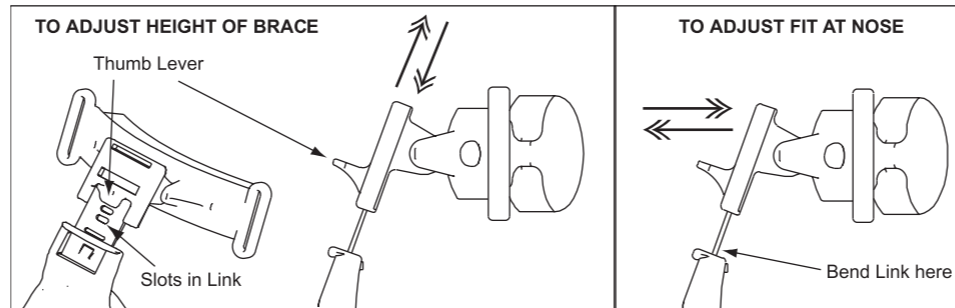**Cleaning Instructions****After Each Use:**

- Remove the headgear from the mask as the headgear will be washed separately. The headgear should be handwashed following the remaining instructions at least every 7 days. Do not machine wash.
- Handwash all components in warm, soapy water. NOTE: Only use pure soap and do not use alcohol, petroleum products, or strong cleaners as these may cause damage and reduce the life of the product.
- Rinse thoroughly and shake excess water from interior surfaces of components. Allow all components to completely air-dry, away from direct sunlight.
- Carefully inspect each component for any wear or damage. Discontinue use if you notice any cracks in the plastic, any tears or hardening of the silicone, or significant fraying or loss of elasticity of the headgear, and contact your healthcare provider for a replacement. Discoloration is normal after several washings and should not be considered sign of wear.

Manufactured for:

30 Spur Dr., El Paso – TX 79906 – USA
Tel: (800) 421-0024 – www.salterlabs.com

Covered by Patents: U.S. 8789532. Other patents pending.
Utopia® is a U.S. registered trademark.

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GaleMed Xiamen Co., Ltd.