

# INTELLICHARGER NEW I2

## USING THE INTELLICHARGER NEW I2 FOR RECHARGING INTUBRITE BATTERIES

The Intellicharger NEW i2 is a universal, automatic smart-charger compatible with almost all types of rechargeable batteries (including the rechargeable batteries supplied with some IntuBrite® laryngoscope handles).

The Intellicharger has the following specifications:

Input : AC 100-240 V 50/60 Hz 0.25 A(MAX) or DC 9 - 12V 1A

Output voltage: 4.36 V ±1% / 4.2±1% / 3.7 V ±1% / 1.48 V ±1%

Output current: 500 mA × 2 / 1000 mA × 1

Dimensions: 5.2" L × 2.7" W × 1.4" H (132 mm × 70 mm × 35 mm)

Weight: 126 g (without batteries and power cord)

It is compatible with the following battery types:

Li-ion / IMR: 10440, 10340, 10350, 12340, 12500, 12650, 13450, 13500, 13650, 14350, 14430, 14500, 14650, 16500, 16340(RCR123), 16650, 17350, 17500, 17650, 17670, 18350, 18490, 18500, 18650, 18700, 20700, 21700, 22500, 22650, 25500, 26500, 26650

Ni-MH / Ni-Cd: AA, AAA, AAAA, C, D

The i2 is restricted to charging Li-ion, IMR, Ni-MH and Ni-Cd rechargeable batteries only. **WARNING:** Never use the i2 for other types of batteries (including alkaline or other non-rechargeable batteries) as this could result in battery explosion, cracking or leaking, causing property damage and/or personal injury.

If you are uncertain about use of a particular battery, you can contact your SunMed® representative, SunMed Customer Care at [customer care@sun-med.com](mailto:customer care@sun-med.com) or SYSMAX Industry Co., Ltd at +86-20-83862000 for further information.

